

PRESENTATION SOFTWARE

Downloading Presentation (For a free, 45-day trial version):

Go to: http://www.neurobs.com/insecure_join/individual

Here, you can sign up for a free account, which you must have even for a demo version.

Once you have registered, log in and go to “Download” on the left-hand column of the site. Download the regular software and it will allow for the 45-day trial.

Installing Experiments –

1. Copy the folder “EFTasks” onto the C:\ drive (or another accessible location)
2. Open Presentation
3. Right click on the white, left-hand column



4. Select “Add Group”

5. Name Group as desired (i.e. Tasks)

6. Right click on the folder

7. Select “Add links to group ‘Tasks’”

8. Navigate to the EFTasks folder on the C:\ drive (or other) and add the .exp file for each of the experiments. Keep in mind that you can create multiple Folders in Presentation to organize different batteries/age groups etc.

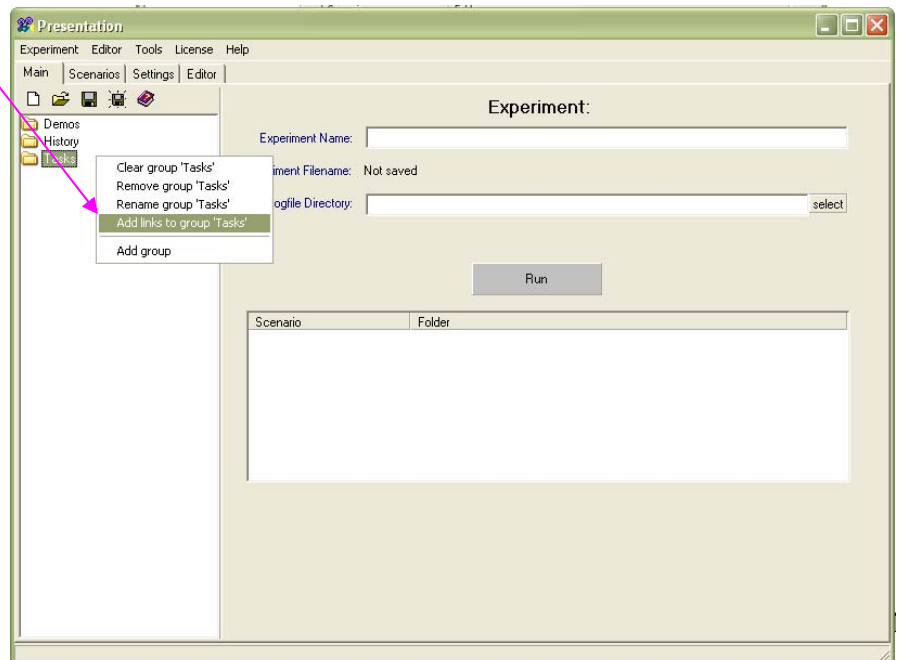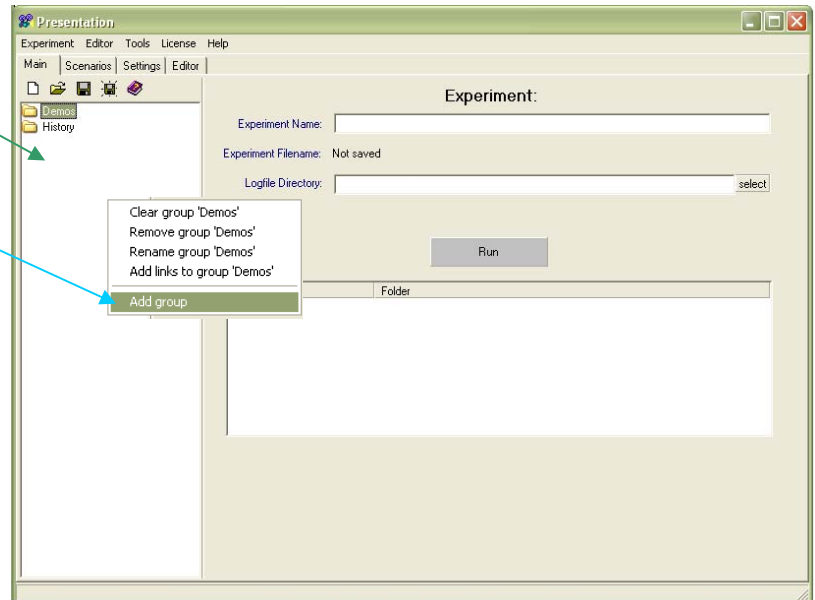


PRESENTATION SOFTWARE

Running Experiments –

1. ****NOTE**** ALL EXPERIMENTS ARE DESIGNED TO RUN AT A RESOLUTION OF 1024 x 768... make sure your monitor is configured to this resolution!!!
2. Open .exp file from the **Experiment** menu or by **double-clicking it in the left-hand column**

3. Under the **“Main”** tab in Presentation, select **“Run”**

4. Enter the **subject ID** and press OK

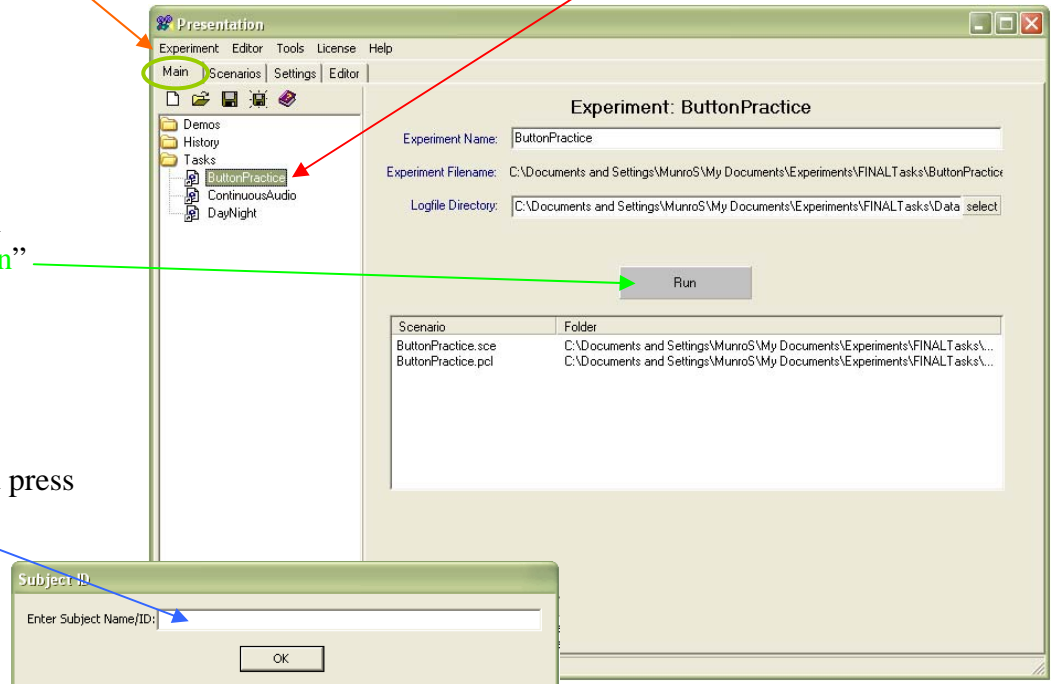

5. Select **“Run Scenario”**
6. When the screen reads **“Ready”** hit <Enter> to begin
7. ****NOTE****: If asked to enter the Subject ID again, DO SO – this is because Presentation stores its own version of events and should be marked by the same ID as the actual data file.
8. ****IMPORTANT**** The experiment **MUST** run to completion! The data file is only saved at the very END of the program and will be lost if you hit Escape during the test.

